

Any-Seal® RC

RESIN-BASED ROOT CANAL SEALER

SHAPING

IRRIGATION

OBTURATION


CALCIUM HYDROXIDE

CORE-BUILD UP

Advantages

- Excellent sealing with optimal flowability
- Excellent biocompatibility
- Excellent radiopacity
- Working time 1hr / 23°C, setting time 3.5hr / 37°C
- Mixing in a 1:1 mixing ratio with dual syringe type
- Low tissue solubility
- Non-staining to the resin


Physical properties

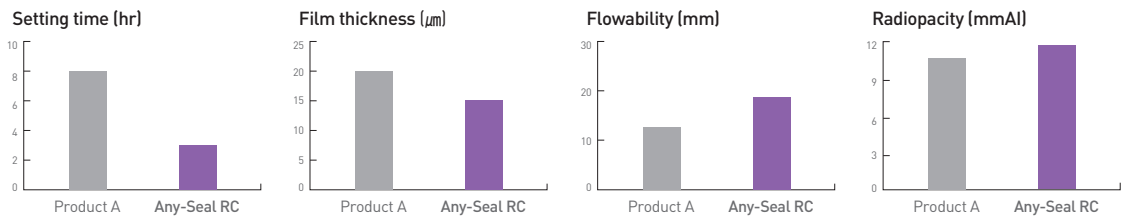


Application




1. Extrude Any-Seal RC connected with auto mixing tip or eco-tip onto the mixing pad at a 1:1 ratio

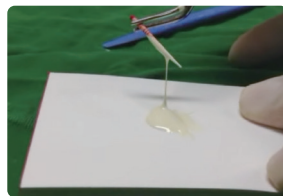

2. Apply Any-Seal RC with GP

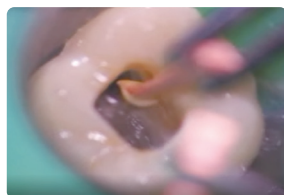


3. Apply into root canal with a proper instrument

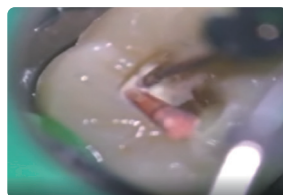

4. Remove the GP from the top of the orifice using a cutter

Results

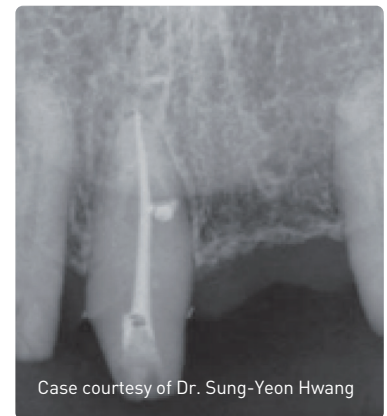

RESIN-BASED ROOT CANAL SEALER

CONTENTS

503-1 Any-Seal RC 10g x 1ea