

Any-Seal[®] RC

A resin-based root canal sealer that provides perfect sealing with excellent flowability and adhesiveness.

Excellent Sealing



Easy & Fast Application

The automixing system provides superior ease of use, offering clinicians a rapid, clean, and perfectly controlled application every time.

Provides stable and reliable sealing




Low solubility with excellent sealing strength. It allows easy removal during re-endodontic treatment

Any-Seal® RC

A resin-based root canal sealer that provides perfect sealing with excellent flowability and adhesiveness

Any-Seal® RC provides exceptional sealing performance with optimum flowability, ensuring a strong and stable bond to the canal walls while minimizing microleakage and reducing the need for retreatment. Its excellent biocompatibility and low tissue solubility support long-term clinical safety and dimensional stability. With outstanding radiopacity, clinicians can easily verify precise obturation on radiographs. In addition, the non-staining formulation ensures that it does not discolor resin materials or surrounding tooth structure. Any-Seal® RC empowers clinicians to achieve superior sealing, excellent handling, and reliable clinical outcomes in every endodontic procedure.

Physical Properties

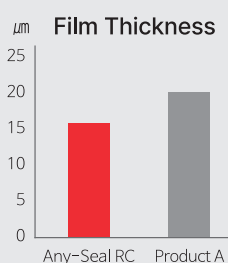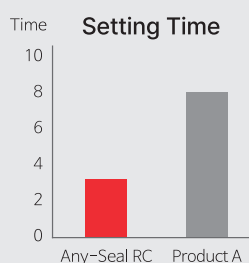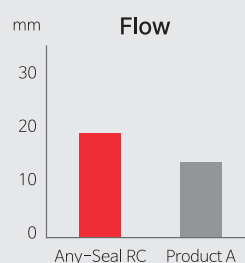

* Source: MEDICLUS R&D




* Dr. Seong-Yeon Hwang,
Saramsarang Dental Clinic

Indications

- Endodontic retreatment
- Internal root resorption
- Permanent obturation of non-vital root canals
- Effective sealing of microleakages in the canal

Radiopacity

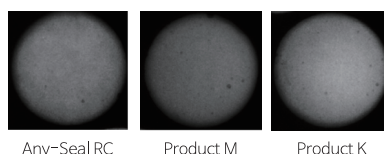


Time

- Working time 1hr / 23°C
- Setting time 3.5hr / 37°C

Application



Extrude Any-Seal RC connected with auto-mixing tip or eco-tip onto the mixing pad at a 1:1 ratio

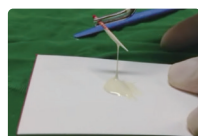

Apply Any-Seal RC with GP



Apply into root canal with a proper instrument



Remove the GP from the top of the orifice using a cutter

Delivery Forms



#503-1 Any-Seal® RC
Any-Seal RC 10g x 1ea

MEDICLUS

Headquarter | No.1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28576, Korea | +82 43 211 2877

Sales Office | No.206 Brown Stone, 464, Cheongpa-ro, Jung-gu, Seoul, 04510, Korea | +82 10 8385 2877

www.mediclus.co.kr | sales@mdclus.com

