


Gingival Retraction & Bleeding Control in One Step

- 
- ✓ **Less time, better 2 in 1 effectiveness**
 - Rapid and reliable bleeding control through the astringent action of aluminum chloride
 - Uniform paste application ensures excellent mucosal adhesion and dispersion
 - ✓ **Comfort for both clinicians and patients**
 - Reduced discomfort, fast and clean rinsing
 - Two viscosities and shades to support treatment strategies

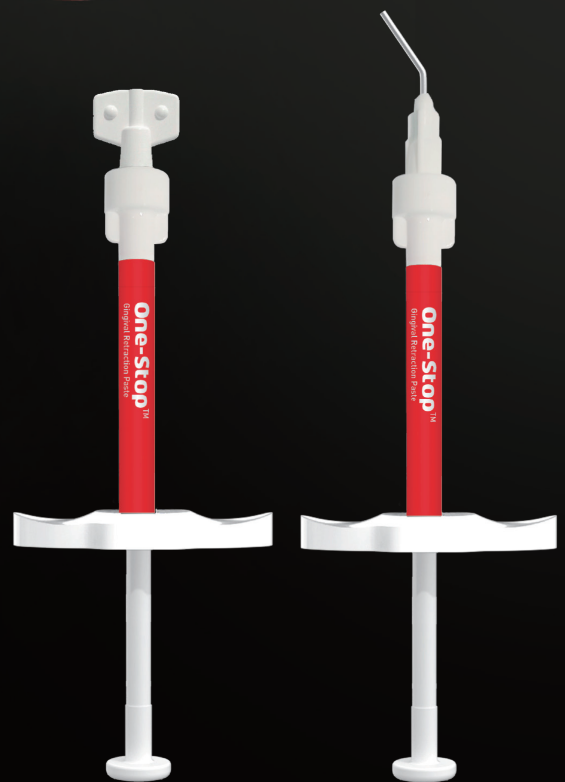

**Stress-Free
Impression
Taking**

NEW

One-Stop™

Gingival Retraction Paste



www.mediclus.co.kr



Video



Catalog

One-Stop™

Fast and simple gingival retraction & Hemostasis with aluminum chloride

✓ **2-in-1** mechanism provides precise gingival management for predictable clinical outcomes

One-Stop™ is a paste-type topical pre-impression agent that provides both gingival retraction and bleeding control simultaneously. With the astringent effect of aluminum chloride, it ensures rapid and stable bleeding control, while its uniform paste consistency allows precise application even in delicate areas. After use, it can be easily rinsed away with water, improving clinical efficiency and enhancing patient comfort.

- ✓ **Indications**
- Gingival retraction and hemostasis prior to impression taking
 - Bleeding control during crown and bridge preparations
 - Moisture and bleeding management before bonding or cementation
 - Gingival management for Class II and V restorations
 - Temporary gingival displacement to improve clinical visibility

✓ **Color**



- **Medium Flow (Yellow)**
- For shallow to medium gingival sulcus
- Expansion and drying of the sulcus prior to digital or conventional impression taking
- Suitable for temporary bleeding control



- **Low Flow (Blue)**
- For deep sulcus or thick gingival tissue
- Recommended as a substitute for the second cord in the double-cord technique

✓ **Step by Step Process**

Clinical Case by Dr. Borikhan Yerborat (KOL)



Before Treatment



Apply One-Stop (2min) & Dry



After Treatment



CONTENTS

- | | |
|---|--------|
| #1101-1 One-Stop™ Blue 0.85g x 7ea | Blue |
| #1101-3 One-Stop™ Blue 0.4g x 30ea (Unit Dose Capsules) | |
| #1102-1 One-Stop™ Yellow 0.85g x 7ea | Yellow |

Seoul Office +82-10-8535-2877

No. 206, 464, Cheongpa-ro, Jung-gu, Seoul 04510, Korea
Fax : +82-2-6499-1006 E-mail : sales@mdclus.com

Head Office Research & development center

No. 1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do 28576, Korea

MDCLUS
MEDICLUS CO., LTD.