

STILL USE SMELLY MATERIAL?

Ezi-Crown®

Light-cured, temporary crown
and bridge resin **FDA**



Single crown




1 Single crown case



2 Dentto-Dam for undercut



3 Apply The-Sep

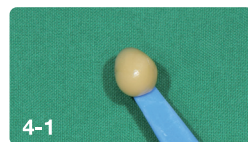

4-1 Taking Ezi-Crown




4-2 Apply Ezi-Crown on the crown



5 Making occlusion and shaping the crown



6 Curing 20 Sec

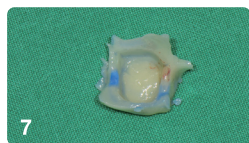

7 Removal & Trimming



8 Fixing with temporary cement

Case by Dr. HoYeol Jang
HoYeol Jang Dental Clinic

Bridge case (Duplication tech)



1 Removal of old fixed bridge



2 Block out for undercut with Dentto-Dam



3 Impression taking for duplication of previous bridge using Any-Flex Clear



4 Removal of old bridge & tooth preparation




5 Filling Ezi-Crown on the tray, deliver it into the mouth and curing the material (20 Sec)



6 Removal of impression tray



7 Trimming

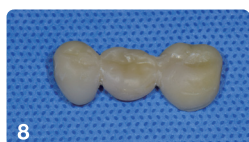

8 Final temporary bridge



9 Fixing on the tooth with temporary cement

Case by Dr. HoYeol Jang
HoYeol Jang Dental Clinic

TEMPORARY CROWN COMPLETED IN LESS THAN 5 MIN.



NO SMELL



SAVE TIME



SAVE COST

Application

1. TEMPORARY CROWN & BRIDGE
2. TEMPORARY PROsthESIS RELINING
3. PROsthESIS DUPLICATION
4. IMMEDIATE IMPLANT RESTORATION

Relining case



1
Temporary implant bridge was not fully fitted by prosthesis error



2
Apply The-Sep



3
Apply Ezi-Crown on prosthesis



4
Fixing on implant



5
Curing and carving of the resin



6
light-curing 10 sec



7
Resin relining on pontic area



8
Result

Ezi-Crown

Contents :

#703-1 Ezi-Crown A1 15g x 2ea

#703-2 Ezi-Crown A2 15g x 2ea

#703-3 Ezi-Crown A3 15g x 2ea

Immediate implant restoration



1
#11 extraction case



2
Impression taking for duplication



3
#11 extraction




4
Implant



5
and suturing



6
Apply Dentto-Dam to protect

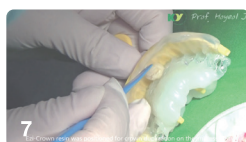

7
Apply Ezi-Crown in tray,
curing 20 Sec



8
Fixing crown with temporary cement

Case by Dr. HoYeol Jang
HoYeol Jang Dental Clinic