

Root Canal Cleaning THE Cleaner

18%

- ✓ Excellent lubrication for instruments during canal shaping
- ✓ Easy removal of the smear layer
- ✓ Decalcifying action inside the canal during shaping
- ✓ Excellent flowability for easy storage and application



Endo@Prep™ EDTA

EDTA Root Canal Cleaning and Preparation



Endo@Prep™ EDTA



✓ Patented Root Canal Irrigant. Proven Results

Endo@Prep™ is available in three formulations — Cream, Gel, and Liquid — allowing practitioners to choose the option that best fits their clinical needs and preferences. The Cream and Gel formulations effectively lubricate files during canal shaping while helping to remove the smear layer.

✓ Indications

- Severely calcified canals
- Enlargement of narrowed canals
- Lubricating effect for files & shaping instruments
- Antibacterial effect within the root canal

✓ Clinical Results

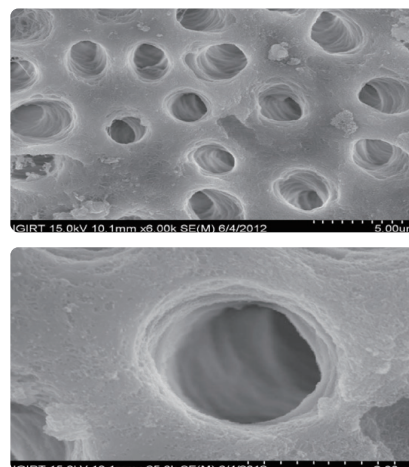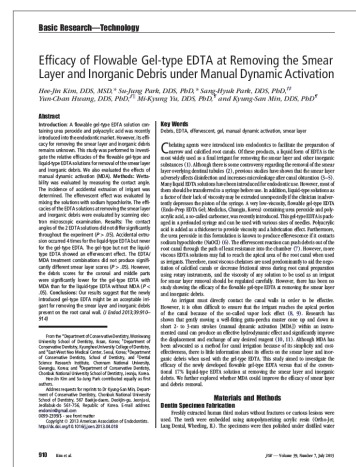


* SEM images courtesy of Prof. Kyung-San Min, Department of Conservative Dentistry, Jeonbuk National University School of Dentistry.

• Published research: “Efficacy of Flowable Gel-type EDTA at Removing the Smear Layer and Inorganic Debris under Manual Dynamic Activation”

— Hae-Jin Kim, DDS, MSD; Sa-Jong Park, DDS, PhD; Sang-Hyuk Park, DDS, PhD; Yun-Chan Hwang, DDS, PhD; Ni-Kyung Yu, DDS, PhD; and Kyung-San Min, DDS, PhD.

✓ Application Process

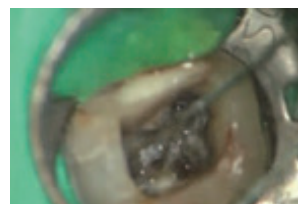

① Apply Endo@Prep™ Gel (10 seconds)



② Rinse with NaOCl



③ Bubbling effect

Gel



Endo@Prep™ Gel
9g x 2ea, Disposable tips

Cream



Endo@Prep™ Cream
9g x 2ea, Disposable tips

Liquid



MD-Solution™
100ml x 1ea

Seoul Office +82-10-8535-2877

Head Office Research & Development Center

No. 206, 464, Cheongpa-ro, Jung-gu, Seoul 04510, Korea
Fax : +82-2-6499-1006 E-mail : sales@mdclus.com

No. 1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do 28576, Korea

MDCLUS
MEDICLUS CO., LTD.